

I-Ring™ Pupil Expander Usage Guide




Introduce the I-Ring into the Anterior Chamber

1 Introduce the Insertor through the primary incision after injecting viscoelastic into the Anterior Chamber. Position the tip centrally over the lens and introduce the I-Ring.




2 Completely retract the prongs (a) before removing the Insertor (b) from the primary incision.

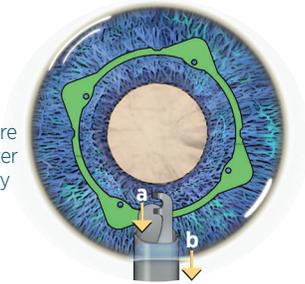


Engage the I-Ring to the Iris


1 Use a Sinsky Hook to secure the distal channel to the iris.

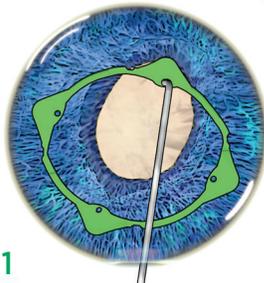

2 Next secure the proximal channel to the iris.

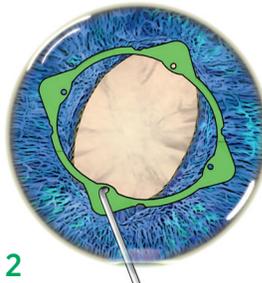


3 Continue to secure the two lateral channels to the iris.

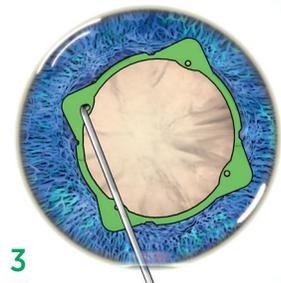


Remove the I-Ring from the eye

1 Use a Sinsky Hook to disengage the distal channel and place it onto anterior iris.

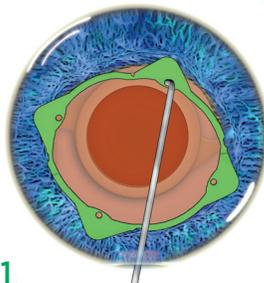


2 Next disengage the proximal channel and place it onto anterior iris. Proceed to disengage the two lateral channels.

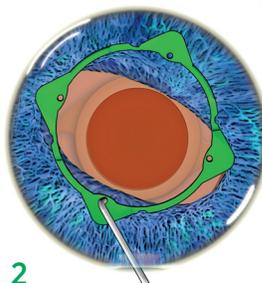


3 With the Insertor in the Anterior Chamber, extend the prongs over the I-Ring before positioning the cannula platform under the hinge.



4 Capture the hinge between the prongs of the Insertor.

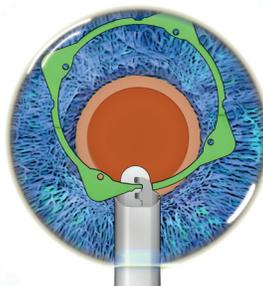

5 Completely draw the I-Ring into the Insertor before withdrawing from the primary incision.

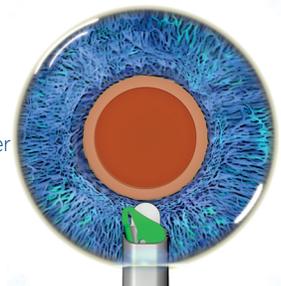

Key:

