


MicroDose™ Injection Kit

NEW

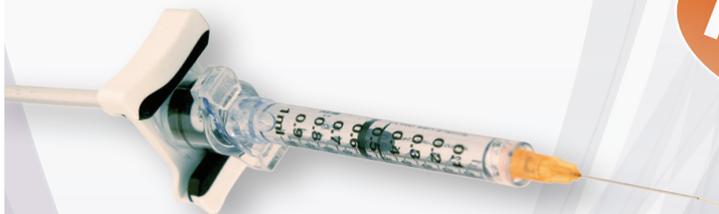

Adapt your vitrectomy console viscous fluid injection set to use a 1mL syringe. Enables full surgeon control for administering subretinal injections with minimal fluid loss.


Place adapter with syringe on VFI tubing connector. Use the console in aspiration mode to draw the injectable into the syringe or inject the fluid directly into the syringe. Place a subretinal injection cannula on the syringe. In VFI mode, use the foot pedal on a low pressure setting (determined by user) to inject as desired.

Kit includes adapter and syringe. MicroDose™ Kit can be connected to the VFI tubing set from Constellation®, Stellaris®, and EVA® systems.

Call to order today!

3275 MicroDose™ Injection Kit

Developed in cooperation with David M. Brown, MD, Houston , TX

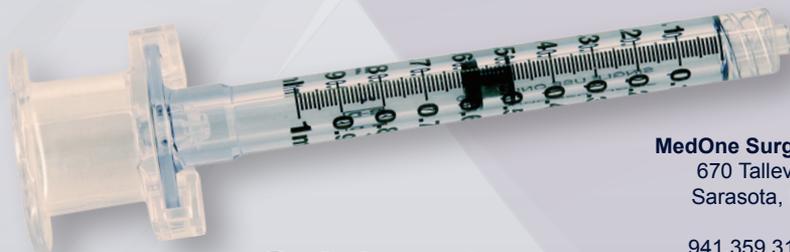

MedOne Surgical, Inc.
670 Tallevast Road
Sarasota, FL 34243

941.359.3129 (TEL)
866.633.6631 (USA)
www.MedOne.com

MedOne@MedOne.com

 **MedOne**
Exactly what you're looking for